# OtoMel®

## NL – OtoMel® oordruppels

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#### Product

OtoMel® oordruppels, flacon 10 ml

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OtoMel® ear drops, 10 ml vial

#### Product description

OtoMel® is a soothing lotion containing pure honey to care for and protect the external ear canal. OtoMel® can be applied when the skin inside the ear is irritated. OtoMel® relieves itching and supports the recovery of the skin barrier. OtoMel® is also suitable for the treatment of eczema.

#### Ingredients

Aqua, Mel, Glycerin, Sorbitol, Potassium Sorbate.

#### Indication

OtoMel® can be applied to care for and protect the external ear canal. OtoMel® can be used for:

- Irritation
- Dry skin
- Eczema
- Itching
- Redness

#### Mode of application

- Heat OtoMel® to body temperature by holding the vial in your hand for several minutes. Using cold ear drops can cause dizziness.
- Apply 1 to 3 ear drops at a time in the ear canal 1 to 3 times a day.
- You can also apply OtoMel® by cutting the gauze to size, rolling it up, and placing it inside the ear. Then you can apply OtoMel® on the gauze and let it be absorbed.

#### Usage during pregnancy or when breastfeeding

There are no indications that OtoMel® should not be used during pregnancy or while breastfeeding.

#### When not to use

- Do not use when you are known to be hypersensitive to honey or one of the other ingredients.
- Do not apply when you have a perforated eardrum caused by, for example, an ear infection.
- Do not use when the packaging is damaged when receiving the product.
- Do not use after the expiry date has passed.
- Do not use for children under 12 years old.

#### Special attention and precautions

- Use for a maximum of 30 consecutive days.

- Cold ear drops can cause dizziness. Therefore, heat the ear drops to body temperature by holding the vial in your hand for several minutes.
- Do not touch the ear with the vial while applying the drops to prevent contamination of the ear drops.
- Re-apply gauze regularly and do not leave it in the ear canal for more than 24 hours.
- Consult your physician if your symptoms do not improve or if you are experiencing new symptoms.
- Keep out of reach of children.

#### Adverse events

- If there are severe adverse events related to the device, these should be reported to the manufacturer and the competent authorities (of the member state) in which the user is located.

#### Storage conditions

- The expiry date is mentioned on the packaging.
- Can be stored for up to 12 months after opening.
- Store between 4°C and 30°C.

#### Date of last revision of the instructions for use

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Dos Medical B.V.
Genex B.V.